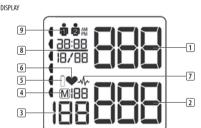
PT | Manual de utilizador

BD-4610

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EN Instruction manual

MEDICAL DISCLAIMER: This device and manual are not meant to be a substitute for advice provided by doctors or other medical professionals

Contact your physician for interpretation of measurements, or if you have or suspect you have a medical issue.

- By ignoring the safety instructions the manufacturer can not be hold responsible for the damage.
- Not for use by children, pregnant women, or pre-eclamptic patients.
- Prolonged over-inflation of the monitor will result in harmful injury to the patient.
- Do not use this device over a wound, as this can cause further injury.
- Too frequent measurements can cause injury due to blood flow interference.
- · Do not attempt to modify this device in any way.
- Never move the appliance by pulling the cord and make sure the cord can not become entangled.
- The appliance must be placed on a stable, level surface.
- This product is for household use only. It is not intended as a substitute for consultation with your doctor.
- For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use. Consult your doctor in case of any doubt.
- The unit should not be operated by children to avoid hazardous situations.
- The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
- · Leaking batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
- If the unit is stored in a cold place, allow it to acclimatize at room temperature before use.
- To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipment's, do not use the device near a cell phone or microwave oven.
- To protect yourself against an electric shock, do not immerse the cord, plug or appliance in the water or any other liquid.
- This appliance shall not be used by children. This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Keep the appliance and its

cord out of reach of children. Cleaning and user maintenance shall not be made by children.

SAFETY BATTERIES

- Do not expose the battery to high temperatures or direct sunlight. Never throw batteries into the fire. There is a danger of explosion!
- Keep batteries away from children. Batteries are not a toy!
- Do not open the batteries by force.
- Avoid contact with metallic objects. (Rings, nails, screws et cetera) there is a danger of short-circuiting!
- As a result of a short-circuit batteries may heat up consider-ably or even catch fire, this may result in burns.
- For your safety the battery poles should be covered with adhesive strips during transport.
- Do not touch a ruptured and/or leaking battery. If the liquid from the battery gets into your eyes, rinse your eyes as soon as possible with clean water, without rubbing your eyes. Immediately go to the hospital. If it is not treated properly, it can cause eye problems.

BLOOD PRESSURE

What is blood pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure

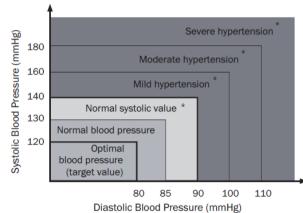
Why measure your blood pressure?

Among the various health problems afflicting modern people. problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying

Blood pressure standard.

The World Health Organization (WHO) and National High Blood Pressure Education Program has developed a Blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard however is a general guideline as individual's blood pressure varies among different eople and different groups...etc

It is important that you consult your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

Blood pressure fluctuation

Blood pressure fluctuates all the time! You should not be worried if you encountered two or three measurements at high levels

Blood pressure changes over the month and even throughout the day depending on circumstances (state of mind, temperature, ...)

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/ stethoscope auscultatory method within the limits prescribed by European Society of Hypertension International Protocol revision 2010 for the validation of blood pressure measuring devices in adults

An irregular heartbeat (IHB) rhythm is defined as a rhythm that is more than 25% slower or 25% faster from the average rhythm detected while the device is measuring the systolic blood pressure and the diastolic blood pressure. When the device detects an irregular rhythm two or more times during the measurement, the Irregular Heartbeat Symbol will be displayed with the measurement values.

PARTS DESCRIPTION

- Display Start/stop button
- Memory button Wrist cuff

Display

- Systolic pressure Diastolic pressure
- Pulse rate
- Memory
- Low battery Pulse detected
- IHB
- Date/time
- User number

BEFORE THE FIRST USE

- · Take the appliance and accessories out the box. Remove the stickers, protective foil or plastic from the device. Battery Installation:
- Pull down on the battery cover, located on left side of this
- 2. Insert two AAA alkaline batteries according to the + and -
- Snap the battery cover on, ensuring it clicks into place. Make sure that the battery cover is securely in position.
- NOTE: When the low battery symbol appears on screen, turn off the device and follow the instructions above to replace batteries

Setting user, year, date and time:

- If device is already on immediately after battery installation toggle between User 1 and User 2 by pressing "start/stop" button. If device is turned off, press and hold "mem" button for 2 seconds, then toggle between user 1 and user 2 by pressing "start/stop" button. To select user, press "mem" to confirm and then flashing year digits occur
- Press "start/stop" to increase one year or press and hold "start/stop" to increase several. Press "mem" to confirm and then flashing month digits occur.
- Press "start/stop" to increase one month or press and hold "start/stop" to increase several Press "mem" to confirm and then flashing day digits occur.
- Press "start/stop" to increase one day or press and hold "start/stop" to increase several Press "mem" to confirm and then flashing hour digits occur.
- Press "start/stop" to increase one hour or press and hold "start/stop" to increase several. Press "mem" to confirm and then flashing minute digits occur.
- Press "start/stop" to increase one minute or press and hold "start/stop" to increase several Press "mem" to confirm and then the device turns off.

Before Taking a Measurement

- · For most accurate results, measurements should be taken in the same circumstances and at the same time every day
- This device can be applied to either wrist, but preferably the left. Use the same wrist for each measureme
- Any blood pressure recording can be affected by the position and physiological condition of the subject, as well as other factors such as background noise, stress, talking and body movement etc.
- Avoid taking measurements while under stress.
- Do not eat, drink alcohol, smoke, exercise, or bathe 30 minutes before using this device, as these activities may change blood pressure.
- For an accurate indication of true blood pressure, take and record several measurements over a period of time. If consecutive measurements are desired, allow two minutes
- between uses to allow blood to flow through the arm normally This device is used to monitor blood pressure. However, it cannot treat or prevent high blood pressure. If you suffer from high blood pressure, consult your doctor. Self-diagnosis and treatment using readings may be dangerous. Follow the

instructions of your physician or licensed healthcare provider.

Measurement

• This device should be used on bare skin. Remove wristwatches or other jewelry. If necessary, loosely roll up your sleeve to expose wrist

- With screen aligned to the midline of your palm and facing towards you, wrap the cuff around your wrist. Secure this device cuff firmly by attaching hook-and-loop fastener
- Comfortably seated, legs uncrossed, feet flat on the floor, back and arm supported, keep the device at the same level as your heart, relax as much as possible and do not talk during the measurement.
- While sitting upright in a chair with your feet flat on the floor press "start/stop". Symbols on screen will flash and cuff will begin to inflate. Do not move, activate arm muscles, or talk during measurement
- Measurement can be stopped at any time by pressing "start/
- When this device senses a pulse, a heart symbol will flash on
- When measurement is complete, the cuff deflates automatically and your systolic, diastolic, and pulse rate will display on screen.
- The screen will change color to indicate the reading. A green screen indicates normal blood pressure, a yellow screen indicates a risk of developing high blood pressure and red indicates high blood pressure. Measurements are automatically stored in the memory.
- Turn off this device by pressing "start/stop", remove from your wrist by detaching hook-and-loop fasteners. If this device is accidentally left on, it will turn itself off after two minutes

NOTE: This device will remain set on the same user after you turn it off. To change to the other user, turn this device off then press and hold "MEM" to access settings. Either the User 1 or User 2 icon will appear. Press "START/STOP" to switch to the other user. Press "MEM" six times until the screen turns off. When you press "START/STOP" again, this device will start as

Using Memory-Recall

This device will store up to 90 measurements each for two users. When more than 90 measurements are taken, the oldest measurement will be replaced

- To access memory-recall, press and release "mem" while the device is off
- First, it will display the average of your three most recent measurements, indicated on screen as M (measurement average).
- Press "mem" again to see the most recent measurement. Press "mem" repeatedly to scroll towards the oldest measurement. On screen, M1 indicates the most recent measurement, with higher M numbers indicating older measurements

If you desire to delete all stored measurements of the current user, press and hold "mem" until "CL" is displayed. Press "start/ stop" to confirm clearing the memory. Press and release "mem" again and a blank entry should appear on screen. Press "start/

Possible reasons Solution

TROUBLESHOOTING

Trouble

Trouble	Possible reasons	Solution	
Err	Error during inflation.	Adjust cuff, ensuring it is wrapped properly, and repeat measurement. Do not move or talk during measurement. Ensure wrist is level with heart.	
Lo	Low battery.	Replace batteries following instructions in the Battery installation section.	
No Power	Low batteries or incorrect battery placement.	Ensure batteries were installed according to the + and - shown on this device. If they were, batteries need replacement.	
Measurement Too High / Too Low	Possible incorrect use of this device.	Adjust cuff, ensuring it is wrapped properly, and repeat measurement.	

CLEANING AND MAINTENANCE

· For regular maintenance, this device only needs to be wiped gently with a soft, dry cloth. Never immerse this device or any components in water

· Do not carry out repairs of any kind yourself. If a defect occurs, please contact your local authorized distributor. Use only authorized parts and accessories.

CALIBRATION AND SERVICE

The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life. It is generally recommended to have this device inspected and calibrated every two years to ensure correct functioning and accuracy Please consult you authorized distributor.

EMC DECLARATION

Guidance and Manufacturer's Declaration - Electromagnetic Immunity, this device is intended for use in the electromagnetic environment specified below. The customer or the user of this device should ensure that it is used in such an environment

Immunity Test	Complianc e	Electromagnetic Environment - Guidance
RF emissions CISPR 11	Group 1	This device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This device is suitable for use in all establishments other than
Harmonic emissions IEC 61000-3-2	Not applicable	domestic and those directly connected to the public low voltage power supply network that supplies buildings used for
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	domestic purposes.

Immunity Test	IEC 60601Test Level	Complia nce Level	Electromagnetic Environment - Guidance
Electrostatic discharge (ESD)IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Power frequency(50 Hz/60Hz)mag netic field IEC 61000-4-8	30 A/m	30 A/m	Magnetic fields from common appliances are not expected to affect the device.
NOTE: LIT is the a.c. mains voltage prior to application of the			


frequency(50 Hz/60Hz)mag netic field IEC 61000-4-8	0074111	00 7 1111	common appliances are not expected to affect the device.
NOTE: UT is the test level	ne a.c. mains	voltage pr	ior to application of the
Radiated RFIEC 61000-4-3	10V/m 80MHz to 2.7GHz 9V/m, 27V/m, 28V/m 385-5785 MHz	10V/m 80MHz to 2.7GHz 9V/m, 27V/m, 28V/m 385-578 5 MHz	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches)to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

SPECIFICATIONS

1	Model	BD-4610
2	Measurement Method	Oscillometric
3	Display	LCD
4	Measurement Range	Pressure: 0 to 299 mmHg Pulse: 40 to 180 beats/minute
5	Accuracy	Pressure: Within ± 3 mmHg Pulse Rate: Within ± 5% of reading
6	Inflation	Automatic inflation by pump
7	Deflation	Automatic rapid deflation
8	Pressure Detection	Semiconductor pressure sensor
9	Memory	2 persons X 90 measurements each
1	Power Supply	2 AAA alkaline batteries (not included)
1	Battery Life	About 6 months when used once a day

1 2	Operating Environment	Temperat ure	50°F (10°C) to 104°F (40°C)	
		Humidity	15% to 90% (noncondensing)	
		Barometri c Pressure	105 kPa to 80 kPa (790 mmHg to 600 mmHg)	
1	Transport Storage	Temperat ure	-4°F (-20°C) to 131°F (55°C)	
	Environment	Humidity	0% to 95% (noncondensing)	
1	Weight of Main Unit	Approximately 95 g (not including batteries)		
1 5	External Dimensions	65 mm (H) X 61 mm (W) X 28 mm (D) (not including the wrist cuff)		
1 6	Measurable Circumference of Wrist	Approximately 5.3" ~ 8.5" (13.5cm ~ 21.5 cm)		
7	Electric Shock Protection	Internal power supply appliance type		

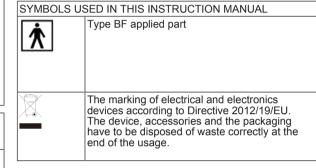Symbols Use

ENVIRONMENT

This appliance should not be put into the domestic garbage at the end of its durability, but must be offered at a central point for the recycling of electric and electronic domestic appliances. This symbol on the appliance, instruction manual and packaging puts your attention to this important issue. The materials used in this appliance can be recycled. By recycling of used domestic appliances you contribute an important push to the protection of our environment. Ask your local authorities for information regarding the point of

BATTERIES

The European directive for batteries (2006/66/EC) states that it is not permitted to throw batteries away with the household waste. They may contain substances which are harmful to the environment. Empty batteries can be handed over to a local, public collection point or a local recycling centre. To avoid overheating as a result of a short circuit, lithium batteries must be removed from the product and the poles must be protected using insulation tape or some other means against short-

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